
Beemoo·CARE

Wearable Flow Electric Breast Pump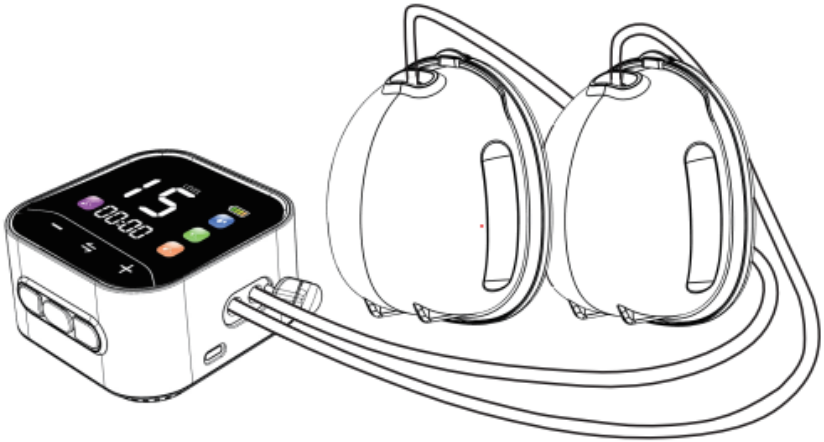

IMPORTANT!

READ CAREFULLY AND
KEEP FOR FUTURE
REFERENCE.

Model: Wearable Flow Electric
Breast Pump (055FF)

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Model: Wearable Flow
Elektrisk Brystpumpe
(055FF)

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WARNING

IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE.

WARNING Do not use this product without reading the instructions for use first.

WARNING Keep the pump and its components out of reach of children – small parts may cause choking and strangulation.

WARNING Do not share the breast pump with others – risk of cross-contamination.

WARNING Do not expose the pump motor to water or other liquids – risk of electric shock.

WARNING Ensure that collected milk does not come into contact with non-sterile surfaces.

- This product is a personal item - do not share.
- Clean and sterilize all parts that come into contact with your breast and breast milk, and inspect for damage before every use.
- Use only the supplied power adapter or charger. Using other equipment may result in electric shock or fire hazard.
- Do not store wet or damp parts, as mold may develop.
- Never pump for longer than the recommended duration – risk of tissue damage.
- Do not store the equipment in direct sunlight or near heat sources.
- Store breast milk according to food safety guidelines – improper handling may pose health risks.
- Only use parts recommended by the manufacturer.
- DO NOT use the product if any part is broken, damaged or missing. Contact Jollyroom AB for replacement parts and instructional literature if needed. DO NOT substitute parts.

BATTERY SAFETY

- Do not disassemble the battery.
- Do not hit, crush, or throw the battery into fire.
- Do not expose the battery to high temperatures.
- Stop using the product if the battery is swollen.
- Do not attempt to replace the battery yourself. It contains flammable and hazardous materials that can cause injury, fire, or explosion.
- Do not insert metal objects into the battery or puncture it.
- Do not charge the battery for longer than necessary.
- Turn off the product before removing the battery.
- The battery must be removed before discarding the product.
- Only authorized service providers may replace the lithium battery. Using the wrong type may cause an explosion.
- Dispose of used batteries according to local environmental regulations.


NOTE! When powered by the built-in battery, the LED screen shows a static battery icon. When connected to the charger and turned off, the LED screen will show a dynamic battery charging animation.

NOTE! When the device is operating and the power adapter is connected, it uses power from the adapter. If the power adapter is disconnected, the device will automatically switch to the built-in battery for power.

NOTE! When the battery is too low, the device will automatically stop working and display a warning to protect the battery. When the battery is fully charged, charging will automatically stop to prevent battery damage.

PARTS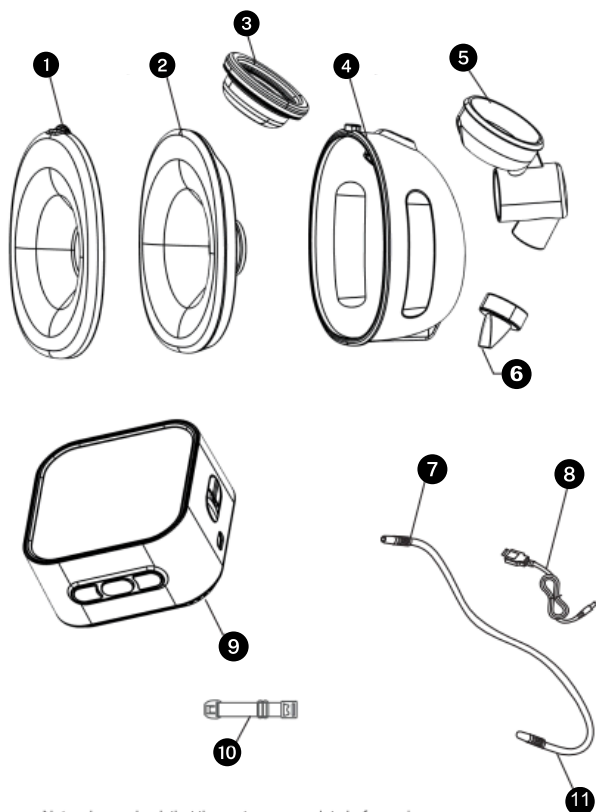

Note: please check that the parts are complete before using.

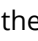
- | | |
|--------------------------|------------------------------|
| 1. Dust cover x2 | 7. Through joint x2 |
| 2. Silicone diaphragm x2 | 8. Charging cable x1 |
| 3. Silicone cylinder x2 | 9. Pump motor x1 |
| 4. 210ml cup body x2 | 10. Bra adjustment buckle x2 |
| 5. Linkers x2 | 11. Flexible pipe x2 |
| 6. Valve x2 | |

CONTROL PANEL

NOTE! This product has a built-in lithium battery. When the battery indicator flashes, we recommend that you use a certified 5V 1A adapter to charge the pump motor as soon as possible.


CONTROL INSTRUCTIONS

- Pumping mode: the  light is on. Pumping mode has 12 intensity levels.
- Express mode: the  light is on. Express mode has 3 intensity levels.
- Massage mode: the  light is on. Massage mode has 10 intensity levels.
- Stimulation mode: the  light is on. Stimulation mode has 15 intensity levels.
- Switch modes by pressing the  button.


NOTE! Fully charge the breast pump before first use. When fully charged, the battery indicator will show a full grid.

ASSEMBLY


NOTE! Before assembling, disinfect and sterilize all components of the pump and thoroughly wash your hands.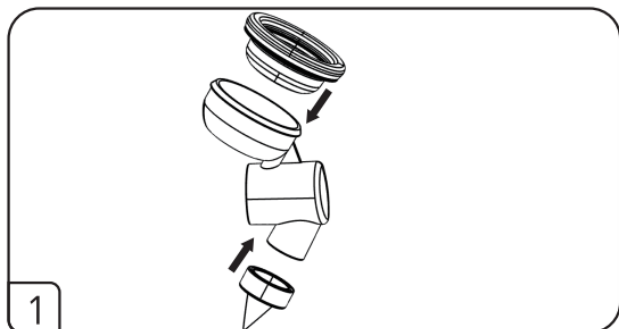


1. Insert the silicone cylinder into the linker and install the valve.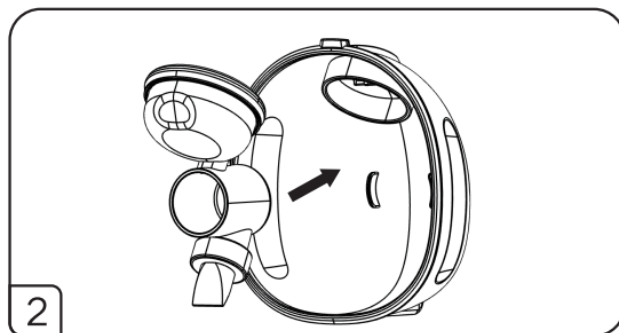


2. Insert the assembled silicone cylinder, linker and valve into the cup body.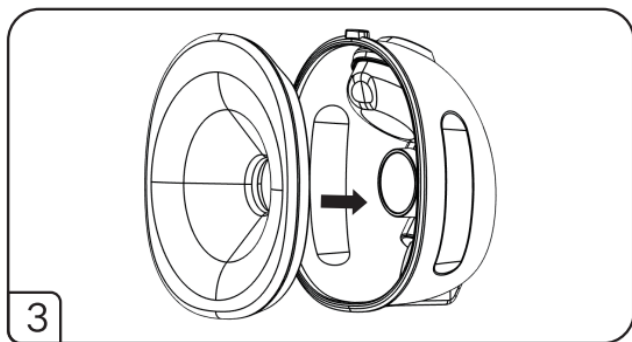


3. Insert the silicone speaker onto the cup body to ensure sealing.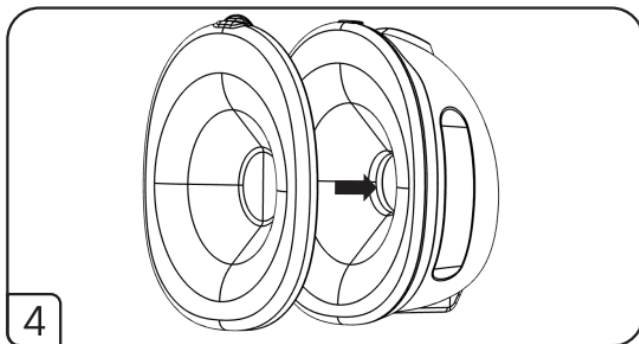

4. Insert the dust cover onto the silicone speaker on the cup body.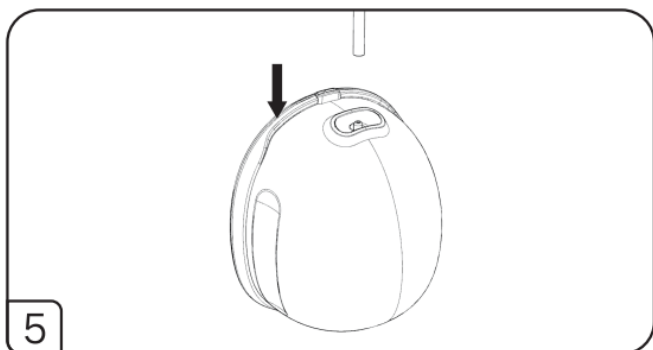

5. Insert the straight end of the hose into the cup body connector.

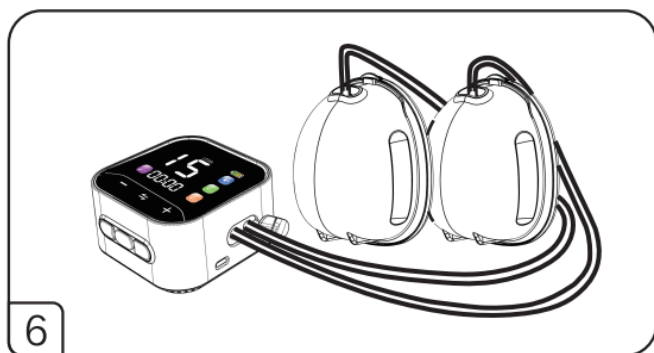6. Insert the straight ends of the hoses into the cylinder head. Make sure the hose aligns with the correct hole position to ensure a secure fit without air leaks.

PUMPING INSTRUCTIONS

NOTE! Wash your hands thoroughly and make sure your breasts are clean.

NOTE! The machine has a memory function. When you turn it on again, the machine will automatically enter the stimulate mode and record the suction levels of each mode.

NOTE! The breast pump will automatically stop after 30 minutes of work.

1. Gently hold the breast, attach the trumpet cover and make sure that your breast is centered, and the massage nursing pad forms a seal against it.
2. Press the on/off button. The pump will start automatically in stimulation mode. Press "+" or "-" to select the massage strength. When milk begins to flow, press the mode button to switch to express mode, then choose the most comfortable suction strength and frequency for pumping.
3. After finishing pumping, turn off the breast pump and carefully remove the main body from the pump.
6. Unplug the main engine component.

SITTING POSITION FOR SUCTION

Relax the body, sit in a comfortable chair and lean slightly forward (use cushion to support back). Make sure to keep the breast pump and milk collector upright.

POURING THE MILK

Keep the breast upright and remove the pump motor from pump body carefully. To pour milk into the bottle, make sure the gap of the milk collector against the bottle mouth.

STORING THE MILK

Store the breast milk in clean and sterilized storage bottle, and put it in the fridge immediately.

NOTE! Only store the collected breast milk in a sterilized breast pump.

NOTE! Do not freeze the milk repeatedly.

NOTE! Do not mix fresh breast milk with frozen breast milk.

DEFROSTING FROZEN MILK

To defrost frozen breast milk, place it in the refrigerator overnight. If you need to use it quickly, you can defrost the milk by placing the bottle upright in a bowl of warm water.

Alternatively, you may use a bottle warmer to heat the milk more quickly.

NOTE! Always ensure the bottle remains upright during warming, and never boil, use a microwave or overheat the milk, as this will cause nutrient loss.

NOTE! Do not refreeze thawed breast milk. Do not mix fresh breast milk with thawed milk.

TROUBLE SHOOTING

1. Lack of or no suction

Cause	Solution
Check whether the valve is correctly installed and not damaged.	Reinstall or change the valve.
Ensure the silicone diaphragm and petal pads are securely attached to the main body of the milk suction device to maintain a proper seal.	Reinstall or change the diaphragm.
Check whether the connection between silicone diaphragm and pump motor is tight.	Tighten all connections.

2. Can not turn on or charge

Cause	Solution
Check whether the output of charger meets the requirements and whether the charger is plugged in properly.	Use the right charger and make sure it is properly plugged in.
The lithium battery has been exhausted.	Charge the pump once low in battery.

3. Milk backflow

Cause	Solution
Diaphragm is not installed properly or is damaged.	Immediately stop pumping and switch off the pump, clean it. Do not use it until you reset the diaphragm or replace with a new.

4. Breast pain when pumping

Cause	Solution
Pain may be caused by excessive force.	Convert the pump to massage mode to create stimulation, and then convert to breast milk mode. Consult a health specialist if problems do not resolve.

TECHNICAL SPECIFICATIONS

Name	Wearable Flow Electric Breast Pump	Expression Frequency	30-50/min
Model No	O55FF	Expression Power	150-270/mmHg
Power Input	5V1A	Massage Power	75-195/mmHg
Battery Capacity	1000mAh	Shut Time	30min
Battery Voltage	3.7V	Noise Limit	≤60dB
Charging Time	2h-3h	Usage Time	1.5h-2h

CARE AND MAINTENANCE

- Clean and disinfect all parts that come into contact with breast milk before and after each use.
- Do not put the motor or power cord in water. Wipe them with a clean, damp cloth only.
- Rinse the valve and suction part gently.
- The suction cup and valve are made of silicone and will wear out over time. Replace when needed.
- Follow the correct order when taking apart or putting the pump back together.
- You may place the bottle, silicone breast shield, valve, and other removable parts in a sterilizer for cleaning.

- Avoid frequent high-temperature sterilization – this can cause them to age or lose shape, which may affect suction. Sterilize no more than 5 minutes each time.
- Do not sterilize the motor or power cord.
- Damaged parts can reduce suction and prevent proper milk expression.
- Do not heat or sterilize the linker in a microwave.

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