Double Electric Breast Pump

IMPORTANT!
KEEP FOR FUTURE
REFERENCE.

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Tuotenumero: 10180280

Instructions for electric breast pump

- Ultraquiet pumping, extremely portable
- Modern design and touch panel for an easy use
- Rechargeable Li-ion battery operated & USB way for a complete flexibility whether at home or on the go.
- It supports single or double pumping switchable

Contents

1. Warnings

2. Functions & Features

3. Technical parameters

4. Assembling & Connection

5. How to use

6. Troubleshooting

1. Warnings

1. The Li-ion battery is built-in, DO NOT remove the battery by yourself.
2. Don't use any adapters or battery that don't come with the pump.
3. Charge the battery once each month. The charging time shall not exceed 6 hours to extend its life time.
4. This is for personal use. Used by more than one person may present a health risk.
5. Always unplug the power chord from the outlet after each use.
6. When cleaning the pump unit, please unplug the adapter.
7. Ensure the tubing is clean without any drops of water.
8. All components can't be sterilized in a microwave or sterilized cupboard.
9. The components (excluding the pump unit and power adapter) are suggested to be sterilized in steam sterilizer or boiled water. They can't be sterilized in a microwave or sterilized cupboard.
10. Don't expose all assemblies with pump unit to high temperature or fire origin and keep it away from children.
11. We reserve the right to product design changes.

2. Functions & Features


1. It's featured by 3-Phase pumping technology.
 - *Stimulation mode imitates fast sucking movements of baby stimulating milk production;*
 - *Expression mode reflects slower, deeper sucking.*
 - *2 in 1 mode combines stimulation with expression 2 in 1, makes it more efficient milk let down.*
- 2.
3. Modern design and touch panel for an easy use.
4. It can be rechargeable battery operated or USB way to make sure you and baby have complete flexibility whether at home or on the go.
5. It perfectly supports single or double pumping switchable.

3. Technical parameters

A/C Adapter	Input 100-240V~50/60Hz.Output 5V~2A
Li-ion Battery Power	3.7V - 2200mAh
Total Power	10W
Stimulation Speed (Single Side)	100 cycles/min
Stimulation Vacuum (Single Side)	-0.013 MPa (Max.), 9 levels adjustable
Stimulation Speed (Double Sides)	60~100cycles/min
Stimulation Vacuum (Double Sides)	-0.013 MPa (Max.), 9 levels adjustable
Expression Speed (Single Side)	26~60 cycles/min
Expression Vacuum (Single Side)	-0.033 MPa (Max.), 9 levels adjustable
Expression Speed (Double Sides)	26~60 cycles/min
Expression Vacuum (Double Sides)	-0.028 MPa (Max.), 9 levels adjustable
Noise	≤60dB (A)
Operating Conditions	5~40°C, 10%~90%Rh
Storage and Shipping Conditions	-25°C ~60°C, 15%~93%Rh
Electronic Protection Equipment Type	Class II

Product description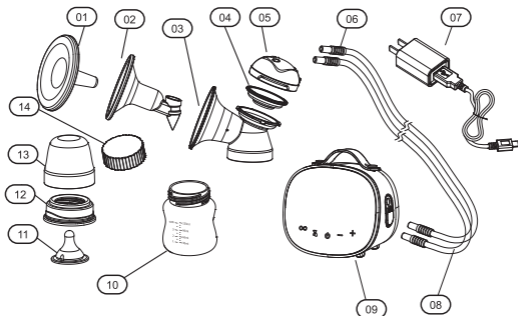

1. Dust cover
2. Silicone breast shield
3. Breast shield body
4. Diaphragm
5. Diaphragm Cap
6. Tubing connector
7. AC adapter
8. Tubing
9. Pump unit
10. Wide neck bottle
11. Nipple
12. Bottle cover
13. Nipple dust cover
14. Bottle Lid



4. Assembling & Connection

Warning

The electric pump comes pre-assembled; however, you need to re-assemble it after each cleaning. Before assembling, the parts which come in contact with breast milk directly, such as milk bottle, silicone breast shield, breast shield body, diaphragm, bottle lid, disc collar (if use standard bottle) must be cleaned and sterilized before each use.

Assembling Expression Collection Kit

Insert the breast shield (02) to the breast shield body (03). You will now have a complete Expression Collection Kit (a). (NOTE: Please remove the dust cover when start pumping (01),)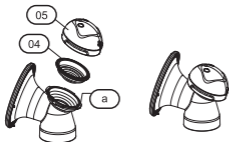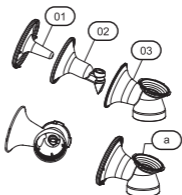

Assembling the Diaphragm

Insert the diaphragm (04) on the top portion of the Expression Collection Kit (a). Screw the cap (05) tightly to prevent air leak as illustrated.

Assembling the Bottle and Bottle Stand



Assemble the assembled breast shield body onto the bottle (10).

Connecting Expression Collection Kit and Pump Unit

1. Connecting Tubing

Connect one end of the Tubing to Expression collection Kit and the other end to the Pump Unit as illustrated. Make sure both ends are connected tightly, otherwise suction may be affected.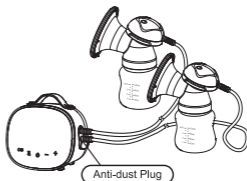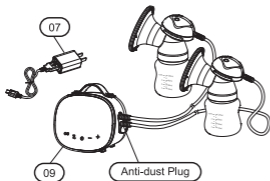

2. Attach the power cord of the power adapter (07) to the pump unit (09) as illustrated, and then plug the power adapter to the socket outlet.


5. How to use

1. Before each use

- Make sure milk collection kit is cleaned and free from any liquid.
- Clean, hot compress and massage the breast with areola, relax yourself to relieve pain resulting from engorged milk.

2. Control panel description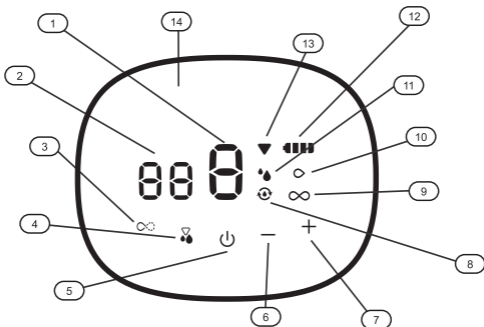

- | | |
|--|-------------------------------|
| 1. Suction Level Display | 8. 2 in 1 |
| 2. Pump Timer | 9. Suction Mode: Double |
| 3. Function Mode: Left, Right or Double | 10. Suction Mode: Left, Right |
| 4. 2 Phase expression selection button adjustment | 11. Expression Mode |
| 5. Pause/Restart (Long press) ON/OFF (Short press) | 12. Battery Power |
| 6. Reduce Vacuum | 13. Stimulation Mode |
| 7. Increase Vacuum | 14. LED Display |

How to use the breast pump:

1. Plug the USB power adapter cord to the power outlet.
2. Place the breast shield over your breast so that your nipple is properly centered in the flange tunnel. Make sure that you have the right flange size that matches your nipple size.
3. Once you are ready, long press the \cup button to switch the device ON.
4. The Breast Pump is a Dual Phase Mode Electric Breast Pump: Stimulation and Expression. Once the device is turned ON, the Timer will count 30 minutes (default). This is the rapid mode designed to mimic your baby's initial feeding motion to stimulate your milk ejection reflex (MER). The breast pump with 3D Pump System has a pumping motion designed to massage, squeeze and suck just like an infant. Press \blacktriangleright to switch from Stimulation to Expression or Expression to Stimulation. You may adjust the suction level according to your need.
5. The Expression Mode is of a slower motion with a deeper sucking motion which is how an infant sucks after the breasts have ejected a good flow of breast milk.
6. Adjust the Suction Level according to what's comfortable for you. Press the plus + button to increase suction level or the minus - button to decrease the suction level.
7. If you want to stop pumping, simply press \cup whatever mode that the pump is working in, the pump will pause working at the current status until you restart pumping. The timer will blink until it restarts.
8. When the timer counts till 30 mins, the pump will stop working automatically.
9. When the pump is working, touch ∞ can switch between single and double pumping.
10. When single pumping, press butt on ∞ and pull out one connector, and then plug the hole with the anti-dust plug tightly.
11. Make sure that you charge the Li-ion battery prior to first use.
12. When the pump is in use through the Li-ion battery power, the Battery Power Indicator $\bullet\bullet\bullet$ will provide a real-time display of your battery status. When the pump is in use through A/C adapter and is in a charging state, the Battery Power Indicator $\bullet\bullet\bullet$ will provide a real-time display of the charging status and the battery power level.
13. When the battery power runs out, the Indicator $\bullet\bullet\bullet$ will flash.

6. Troubleshooting

If there is low or no suction

- Check **all** connections **if** they are **all** attached at **all** connections points properly and **tightly**.
- Make sure the tubing isn't bent, or without any **blockage** in it.
- Make sure the **silicone** breast shield body and the **plastic** accessories are not deformed **alter** the steam sterilization.

The pump can not switch on

- Check if the power chord is properly attached to the pump.
- Ensure the battery power is **available** without the adapter.

Milk backflow

- Check if the power chord is properly attached to the pump.
- Ensure the battery power is **available** without the adapter.

Anything wrong with this breast pump, please contact your dealer or the store where you buy this breast pump.

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Jollyroom AB