Single Electric Breast Pump

IMPORTANT!
KEEP FOR FUTURE
REFERENCE.

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Tuotenumero: 10180281

Instructions for electric breast pump

- Ultra quiet pumping, extremely portable
- Modern design and touch panel for an easy use
- Rechargeable Li-ion battery operated & USB way for a complete flexibility whether at home or on the go.

Contents

- | | |
|-------------------------|----------------------------|
| 1. Warnings | 4. Assembling & Connection |
| 2. Functions & Features | 5. How to use |
| 3. Technical parameters | 6. Troubleshooting |

1. Warnings

1. The Li-ion battery is built-in, DO NOT remove the battery by yourself.
2. Don't use any adapters or battery that don't come with the pump.
3. Charge the battery once each month. The charging time shall not exceed 6 hours to extend its lifetime.
4. This is for personal use. Used by more than one person may present a health risk.
5. Always unplug the power chord from the outlet after each use.
6. When cleaning the pump unit, please unplug the adapter.
7. Ensure the tubing is clean without any drops of water.
8. All components can't be sterilized in a microwave or sterilized cupboard.
9. The components (excluding the pump unit and power adapter) are suggested to be sterilized in steam sterilizer or boiled water. They can't be sterilized in a microwave or sterilized cupboard.
10. Don't expose all assemblies with pump unit to high temperature or fire origin and keep it away from children.
11. We reserve the right to product design changes.

2. Functions & Features


1. It's featured by 3-Phase pumping technology.
 - *Stimulation mode imitates fast sucking movements of baby stimulating milk production.*
 - *Expression mode reflects slower, deeper suckling.*
 - *2 in 1 mode combines stimulation with expression 2 in 1, makes it more efficient milk let down.*
2. Ultra quiet pumping, extremely portable.
3. Modern design and touch panel for an easy use.
4. It can be rechargeable battery operated or USB way to make sure you and baby have complete flexibility whether at home or on the go.
5. Mute button to mute the pressing noise not to disturb the sleeping baby.

3. Technical parameters

Power Adapter	
Power Input	100-240V ~ 50/60Hz
Power Output	5V 2A
Power Rating	10 W
Lithium Battery	3.7V 2200mAh
Stimulation Speed	100T/min
Expression Speed	26-60T/min
Stimulation Vacuum	-0.002 ~ -0.019Mpa(max), 9 levels vacuum
Expression Vacuum	-0.013 ~ -0.036Mpa(max), 9 levels vacuum
Timer	30mins
Overall Noise	<60dB(A)
Operating Conditions	5 - 40°C, RH ≤90%
Electric Shock Protection	AC adapter Class II, Host machine Class III

Product description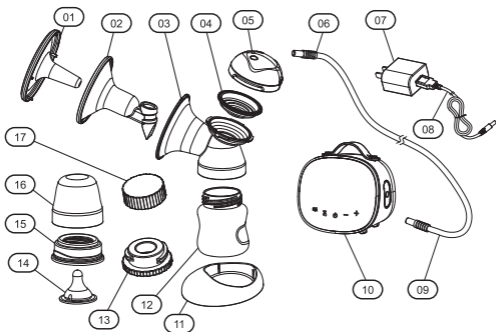

- | | | |
|------------------------|----------------------|-----------------------|
| 1. Dust cover | 7. Power Adapter | 13. Bottle Adapter |
| 2. Silicone 3D flanges | 8. Adapter Cable | 14. Nipple |
| 3. Breast Shield Body | 9. Tubing | 15. Disc Collar |
| 4. Diaphragm | 10. Pump Unit | 16. Nipple Dust Cover |
| 5. Cap | 11. Bottle Stand | 17. Bottle Lid |
| 6. Tubing Connector | 12. Wide Neck Bottle | |



4. Assembling & Connection

This electric breast pump comes pre-assembled, however you need to re-assemble it after each cleaning. Before assembling, parts which directly get in touch with breast milk, such as milk bottle, Silicone 3D flanges including, breast shield body, diaphragm, bottle lid, disc collar (when using standard bottle) must be cleaned and sterilized before each use.

Assembling milk collection kit

1. Assembling expression collection kit

Insert the silicone 3D flanges (02) to the breast shield body (03). You will now have a complete Expression Collection Kit (a). Note: Please remove the dust cover when start pumping.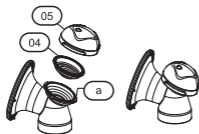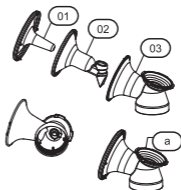


2. Assembling the diaphragm

Insert the diaphragm (04) on the top portion of the Expression Collection Kit (a). Screw the Cap (05) tightly to prevent air leak as illustrated.

3. Assembling the Bottle and bottle stand


Assemble the assembled breast shield body onto the bottle (12). Affix the assembled collection kit to the bottle stand (11).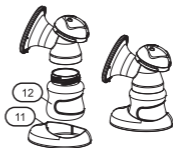

Connecting milk collection kit and pump unit

1. Connecting the pump unit to milk collection kit

Insert the tubing connector (06) at both ends of the tubing (09) into the air vents of the diaphragm cap (05). Insert the other side of the tubing into right and left side of the pump unit (10).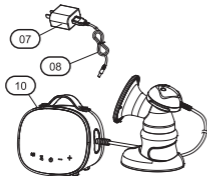

2. Connecting the pump unit to the power supply

Connect the cable (08) to pump unit (10), then plug the adapter (07) into an electrical socket. The pump unit enters the standby mode.


5. How to use

1. Before each use:

- Make sure milk collection kit is cleaned and free from any liquid.
- Clean, hot compress and massage the breast with areola, relax yourself to relieve pain resulting from engorged milk.

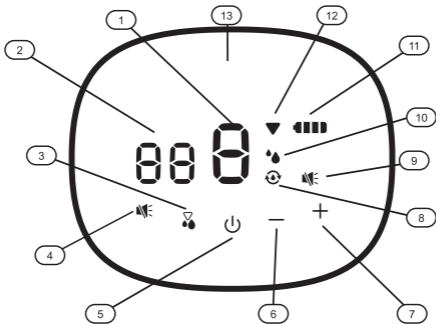2. Control panel description:

1. Vacuum Level
2. Timer
3. Phase button: Stimulation/Expression/2 in 1 (9 levels adjustable)
4. Mute button pressing noise
5. Long press to switch on or off; Short press to pause or resume
6. Reduce vacuum
7. Increase vacuum
8. 2 in 1
9. Mute
10. Expression mode
11. Battery reminder
12. Stimulation mode
13. LED display

3. ON/OFF button:

- Long press the  button to turn on or turn off the pump unit; It enters Stimulation mode Level 1 by default.
- Short press  button to pause or resume the working status.

4. Pumping mode and vacuum level adjustable:

- Under the Stimulation mode, 9 vacuum levels adjustable by pressing + or –
- Under the Expression mode, 9 vacuum levels adjustable by pressing + or –
- Under the 2 in 1 mode, 9 vacuum levels adjustable by pressing + or –
- Switch the pumping mode by  button.
- Short press  button to mute the button pressing sound. The cycle will get lower at the mute state, but the vacuum is not affected.

5. LED screen display:

- Under the Stimulation mode,  icon is lit; Under the Expression mode,  icon is lit; Under the 2 in 1 mode,  icon is lit.
- The number of 8 shows the vacuum level.
- When the AC adapter is not connected, the pump automatically uses the built-in Lithium battery power supply, the  icon lights up and displays the current power in real time.
- When the battery is nearly exhausted, the  icon will flash and beep. LED screen is off in 10s.

6. Timing Function:

- When you long press  to start pumping, 30mins timer will begin; The pump will stop working, and then enters the standby status till 30mins timer is done.
- In pumping status, if you long press ; the pump will stop working, the timer will be cleared to 00min automatically; If you long press  again, new timer will re-start from 00min.

7. Lithium battery supply :

- In working status, the pump unit is powered when the AC adapter is connected; When the AC adapter is not connected, the pump automatically uses the built-in Lithium battery power supply.
- In the shutdown state, connect the power adapter to charge the lithium battery; After fully charged, it can work continuously for around 200mins.
- When the battery power is too low, the pump unit automatically stops working and alarms to protect the Lithium battery; When the battery is fully charged, it will stop automatically to avoid the damage to the battery.

6. Troubleshooting

If there is low or no suction

- Check **all** connections **if** they are **all** attached at **all** connections points properly and tightly.
- Make sure the tubing isn't bent, or without any **blockage** in it.
- Make sure the **valve** body with membrane and the **plastic** accessories are not deformed **alter** the steam sterilization.

The pump can not switch on

- Check it the power chord is properly attached to the pump.
- Ensure the battery power is available without the adapter.

Milk backflow

- **If** milk or water is found in the **bladder** or tubing, **please** rinse the tubing with **clean** hot water. Hang to **AIR DRY COMPLETELY BEFORE USING**. You can **also** check if the **bladder** is good but not broken.

Anything wrong with this breast pump, please contact your dealer or the store where you buy this breast pump.

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Jollyroom AB